

Essential Information to know: ESSENTIAL INFORMATION




Multifuntion Port Change to ENFit

Enfit compatible Multifunction Port Will be in stock on March 12th.

The new **ENFit Enteral Syringe** easily connects to the Kangaroo Multi-functional Port via **ENFit connection**, to reduce the risk of enteral tube feeding misconnections and improves patient safety.

Steps:

- Verify feeding tube or access device/extension set is **ENFit** compatible.
- Proceed to thread the **ENFit syringe** to the **Multifunction Port**.
 - ⇒ When attaching the **ENFit syringe** to the **ENFit Transition connector** be sure the syringe tip is free of medication or debris.
- The **ENFit syringe** and connector are now ready to be used on the **Kangaroo Multi-functional Port** or PEG tubing.

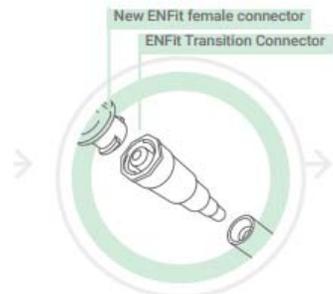

Important Information:

Central Logistics carries **Kangaroo Multi-functional Port with Single Lumen Adapter** which make it possible to connect the **Kangaroo Multi-functional Port** to a PEG Tube.

Steps:

- Call Central Logistic and request a **Kangaroo Multi-functional Port with Single Lumen Adapter**
- Open the package containing both the **Kangaroo Multi-functional Port** and the package containing the **Single Lumen adapter**
- Apply the **Single Lumen Port adapter** to the bottom (the double sided prong) of the **Kangaroo Multi-functional Port**
- The tubing is now ready to connect to the **non ENFit compatible PEG Tube**.
- Soon you will begin seeing **ENFit compatible PEG Tubes** inserted



All **ENFit Syringes, ENFit Transitional Connectors, and Kangaroo Multi-functional Ports with Single Lumen Adapter** are available for patient use, and located on the unit or Central Logistics.